

ALDEVON™

Cleanser
Deodorizer-Mildewstat
Aldehyde-Free Surface & Environment Cleanser



ALDEVON is recommended in high risk areas like
ICU, ICCU, SICU, PICU, NICU, MICU, TRAUMACARE, BURN WARDS,
MINOR OT, MAJOR OT, AKD, PHARMA STERILE SECTION LABOUR ROOM etc.

ALDEVON™

Cleanser Deodorizer-Mildewstat

Aldehyde-Free Surface & Environment Cleanser

ALDEVON as broad spectrum activity virucidal, bactericidal, fungicidal tuberculocidal, sporicidal, MRSA, etc. 4th Generation quaternary ammonium compound.

ALDEVON Composition :

Polyhexamethylen Bigunides 2.4% >, Polyhexamethylen Guinidine 2.5% >

Alkeyldimethylbenzylammonium chloride 15% >

Didecyldimethyl benzylammonium chloride 8.5% >

Nonionic Surfactants & Inbuilt corrosion inhibitors etc

ALDEVON is very effective on porous & nonporous surfaces, excellent residual action. rapid action begins within, 10 min to 15 min and cleans the surface without any damage, biodegradable in nature non irritant & nontoxic.

ALDEVON acts on DNA & RNA crosses cell wall membrane and destroys the material responsible for cell replication.

Disinfection of Instruments immerse Aldevon 20ml in one Lt cold water for 30 minutes ALDEVON 20 ml diluted in cold water for all thermolable & thermo stable instruments used in clinical practice can be immersed for 30 min which removes Lipids & proteins effectively without damaging the instruments as per the instruction given (ALDEVON 2% Dilution for instruments) 20 ml in 1Lt cold water. ALDEVON usage method shake the jar and pour the required solution in a clean bucket pour 5 Lt water and dilute the solution of 50 ml for noncritical area & 75 ml for critical care area take a clean mop use the twin bucket system (in one bucket clean water & other bucket ALDEVON diluted solution) while mopping switch off all the electrical gadgets including AC through mopping is required and the chemical should be in contact with the surface till it dries on its own for optimum results. Always prepare fresh solution for mopping or fumigation.

Precaution Measures: Keep away from children, avoid direct contact with concentrated liquid do not mix with other liquid. Use gloves while mopping & wiping the walls surface equipments like trolley, table, lights, etc .

	Vol. of ALDEVON	Vol. of Cold Water	% Dilution
1. FUMIGATING	3.5 ml.	1 Lt.	0.35%
2. WIPING / MOPING (General Areas)	10 ml.	1 Lt.	1%
3. WIPING / MOPING (ICU, SICU, NICU, Burns Ward, Etc.)	15 ml.	1 Lt.	1.5%
4. CLEANSER (All instruments)	20 ml.	1 Lt.	2%



Manufactured by:



Medichem Products Pvt. Ltd.
No. 79/181, Industrial Estate
Masat DNH-Silvaasa

Exclusive Mktg.

By :



**absolute
solutions**
Bio-Security Solutions

+91 9825038033

+91 9408621326

FOR EXTERNAL USE ONLY